



### The Jumper JPD-FR202 Infrared Thermometer

The image on the left shows the IR Sensor and the Power Button.

The image on the right shows the display and three buttons, from left to right they are: the C/F Button that will change the reading from Celsius to Fahrenheit, the M Button which stands for Mode (there are two modes, the first is the Object temperature mode which is denoted by the image of a house in the left hand corner of the display. The second is the Forehead temperature which is denoted by the image of a person's head in the center of the display. The Third button is the mute or un-mute button.

Before completing the temperature screening, make sure that the C/F button is set to F, for Fahrenheit and that the Mode button is set to Forehead (denoted by the image of a person). This will ensure a proper scan.